

Wheelchair Basket Retrofit Instructions



Components: basket, sign, (4) clips for sign, bracket and spring, arm bracket with rod, arm bracket with jacknut, (4) Black screws long, (6) black washers, black screw short, basket clamp, (4) carriage bolt, (4) nylon lock nut, shoulder bolt, and nylon lock nut.



Remove left arm pad and install arm bracket under arm pad with rod facing up as shown. Use two black washers on each long black screw to attach arm pad.



Remove arm pad and install bracket with jacknut installed as shown. Use one black washer on each long black screw to attach arm pad.

For 20" chairs no washer is needed



Attach basket using basket clamps with carriage bolts. The picture is the approximate location for 24" chairs, the red "X" is for 22" and the green "X" is for 20". Only hand tighten the nuts at this point. Attach sign in upper corner as shown.



Slide spring and bracket over the rod on the right arm of the basket assembly. Attach the assembly to the bracket under the arm pad using the screw provided.



Continue to push tube in and align the end of the spring with the opening on the tab welded on the arm.

Questions: email customerservice@myamigo.com or call 1-800-248-9131 or 989-777-0910

Wheelchair Basket Retrofit Instructions



After aligning spring in tab, rotate the basket and latch the far side tab into the red rod on the far side.



Install shoulder bolt and screw as shown.



Bracket latch should align over rod on left arm pad. If alignment is off sit in chair and shift basket straps until alignment is correct and tighten nuts. If alignment is correct tighten nuts and check fitment again.



Questions: email customerservice@myamigo.com or call 1-800-248-9131 or 989-777-0910